



Fecal Swab

Sterilin Fecal Swab

- Designed for the collection, transportation and preservation of faecal samples
- Very versatile - format is ideally suited to automation and standard culture
- Capture cap design facilitates plating during manual bacterial culture
- Transport medium is semi-liquid Cary Blair – suitable for the transport of enteric bacteria
- Aids easy and speedy patient diagnosis

Instructions for use

1. Open the pouch containing the Fecal Swab
2. Remove the tube and swab and collect the specimen either from rectal canal or stool sample
3. Unscrew cap and insert swab into the tube
4. Break the applicator at the point indicated by the coloured line
5. Replace cap on tube and screw tightly. The 'capture cap' design will then locate the swab so that it becomes integral within the cap – ideal for plating during manual bacterial culture.



G&N Laboratory Ltd
Stane Street
Slinfold, Horsham
West Sussex
RH13 0TW

Tel: (0)845 2638908
sales@gandn.com



Sterilin Limited
Angel Lane
Aberbargoed Bargoed
Caerphilly CF81 9FW
United Kingdom

Customer services:
Tel +44 (0)844 844 3737
Fax +44 (0)844 844 2373

Head office:
Tel +44 (0)1443 830830
Fax +44 (0)1443 821545
www.sterilin.co.uk



Technical data and ordering information

Product Code	Description	Case Qty
470CE.A	Faecal swab kit – tube + 2ml semi-liquid Cary Blair + regular flocked swab	50

All enquiries to:

