



UriSwab

Sterilin UriSwab

- For the collection, transport and preservation of urine samples
- Ideal for use in automation
- Simplifies collection of urine samples: unique sponge swab is impregnated with the exact amount of boric acid for the sample collected - no fill line required
- Available in rigid or squeezable tube formats

Directions for Use

1. Unscrew tube and remove UriSwab applicator
2. Either hold applicator sponge in urine flow or dip sponge into urine sample collected for five seconds to saturate
3. Return UriSwab applicator to tube and secure cap. Send sample to the laboratory for processing
- 4 & 5. In the laboratory centrifuge tube to release sample into the base of the tube.
Alternatively, when using the squeezable UriSwab simply squeeze the tube to release the sample from the sponge.



G&N Laboratory Ltd
Stane Street
Slinfold, Horsham
West Sussex
RH13 0TW

Tel: (0)845 2638908
sales@gandn.com

Sterilin
Limited

Sterilin Limited
Angel Lane
Aberbargoed Bargoed
Caerphilly CF81 9FW
United Kingdom

Customer services:
Tel +44 (0)844 844 3737
Fax +44 (0)844 844 2373

Head office:
Tel +44 (0)1443 830830
Fax +44 (0)1443 821545
www.sterilin.co.uk

Sterilin **IWAKI**

Sterilin **Sterilin**
PACKAGING PRODUCTS SILICONE DEVICES

Technical data and ordering information

Product Code	Description	Case Qty
802CE.A	Sterilin UriSwab	300
800C	Sterilin UriSwab, squeezable	100

All enquiries to:

G+N LABORATORY