



Griff® DAILY Sharps container


Assembly and usage instructions

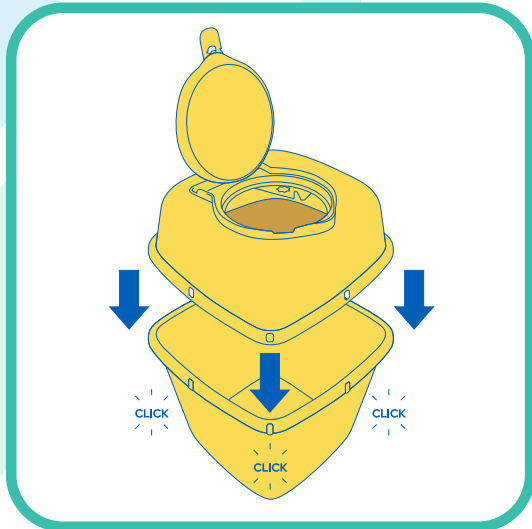


- 1** Fix the lid to the base by applying firm pressure. You'll hear and feel clicks as the lid fixes into place.

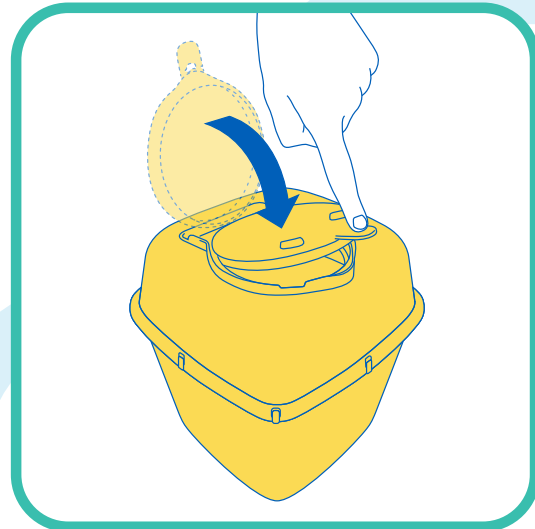

- 2** For temporary closure, apply light pressure to lid where shown, until you hear and feel a click.

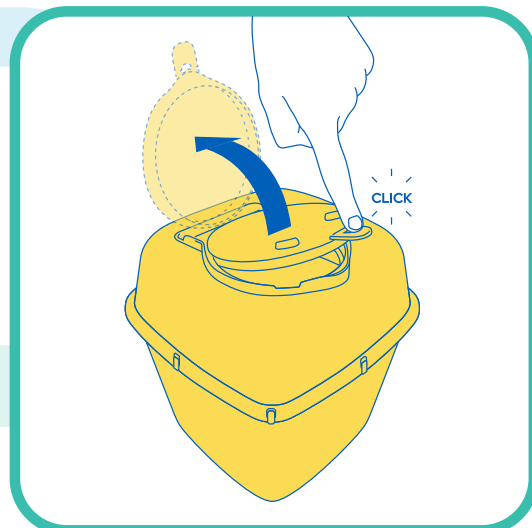

- 3** To release temporary closure, press the end of the tab down.

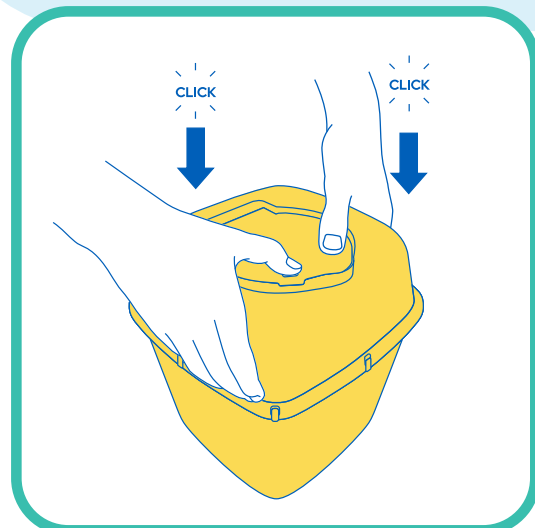

- 4** To permanently seal, place your thumbs on the indentations and push firmly until two clicks are heard and felt. A final visual check is recommended.

**Do not fill above the indicated line.
Complete the relevant sections of the label on the container.**