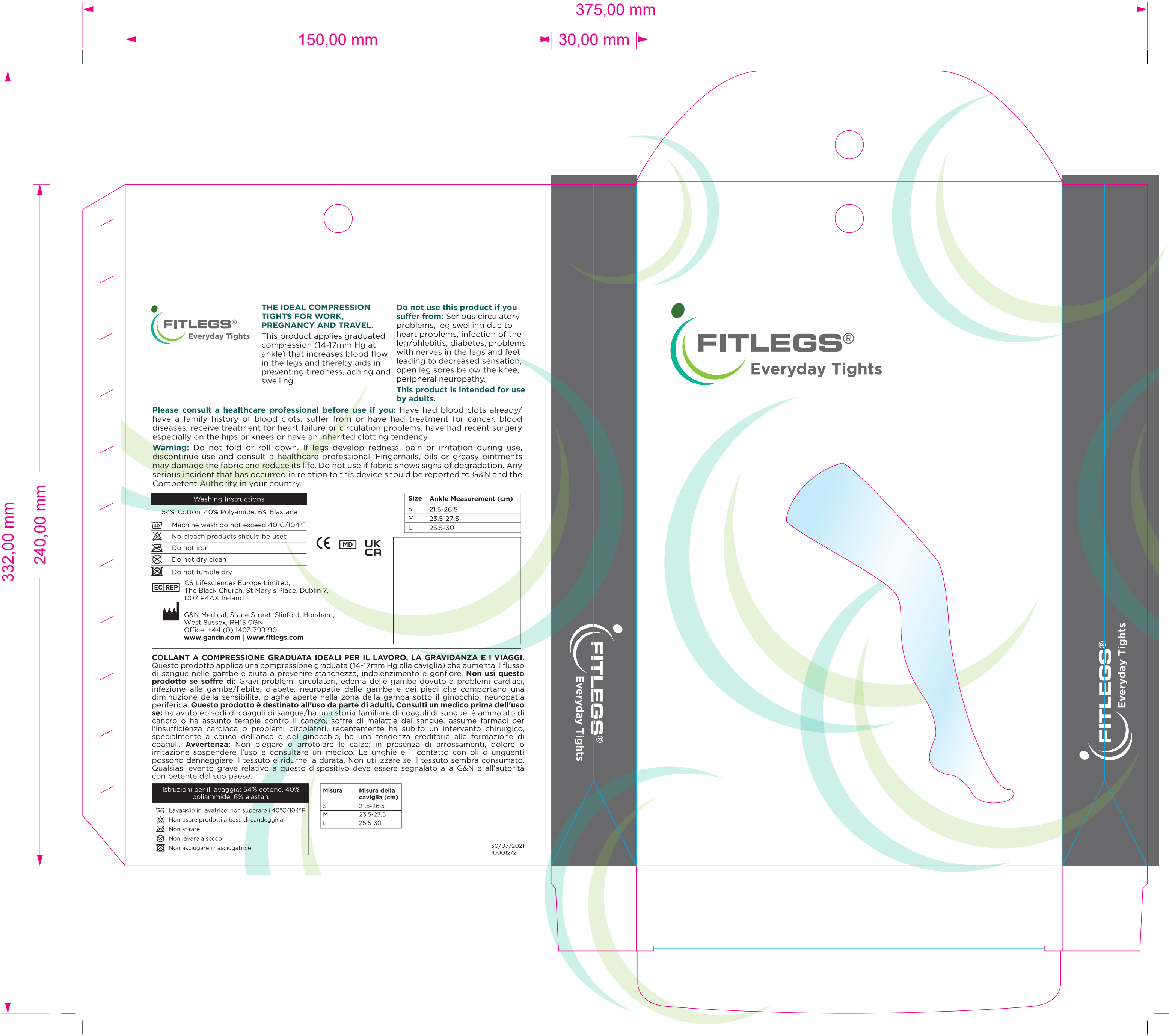
**THE IDEAL COMPRESSION
TIGHTS FOR WORK,
PREGNANCY AND TRAVEL.**

This product applies graduated compression (14-17mm Hg at ankle) that increases blood flow in the legs and thereby aids in preventing tiredness, aching and swelling.

Do not use this product if you suffer from: Serious circulatory problems, leg swelling due to heart problems, infection of the leg/phlebitis, diabetes, problems with nerves in the legs and feet leading to decreased sensation, open leg sores below the knee, peripheral neuropathy.

This product is intended for use by adults.

Please consult a healthcare professional before use if you: Have had blood clots already/ have a family history of blood clots, suffer from or have had treatment for cancer, blood diseases, receive treatment for heart failure or circulation problems, have had recent surgery especially on the hips or knees or have an inherited clotting tendency.

Warning: Do not fold or roll down. If legs develop redness, pain or irritation during use, discontinue use and consult a healthcare professional. Fingernails, oils or greasy ointments may damage the fabric and reduce its life. Do not use if fabric shows signs of degradation. Any serious incident that has occurred in relation to this device should be reported to G&N and the Competent Authority in your country.

Washing Instructions

54% Cotton, 40% Polyamide, 6% Elastane

Machine wash do not exceed 40°C/104°F

No bleach products should be used

Do not iron

Do not dry clean

Do not tumble dry

CS Lifesciences Europe Limited,
The Black Church, St Mary's Place, Dublin 7,
D07 P4AX Ireland

G&N Medical, Stane Street, Slinfold, Horsham,
West Sussex, RH13 0GN
Office: +44 (0) 1403 799190
www.gandn.com | www.fitlegs.com

Size	Ankle Measurement (cm)
S	21.5-26.5
M	23.5-27.5
L	25.5-30

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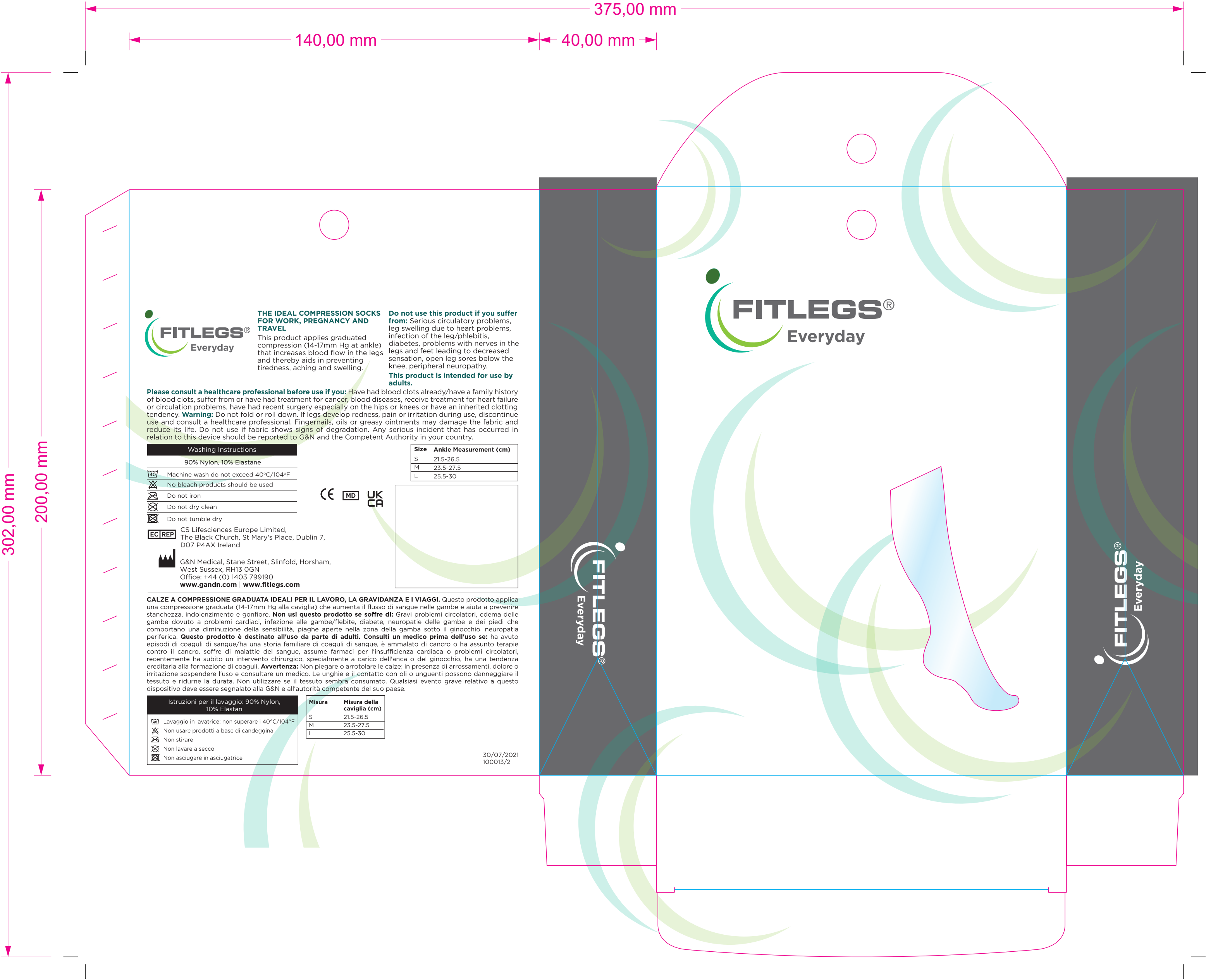
30/07/2021
100012/2

THE IDEAL COMPRESSION SOCKS FOR WORK, PREGNANCY AND TRAVEL

This product applies graduated compression (14-17mm Hg at ankle) that increases blood flow in the legs and thereby aids in preventing tiredness, aching and swelling.

Do not use this product if you suffer from: Serious circulatory problems, leg swelling due to heart problems, infection of the leg/phlebitis, diabetes, problems with nerves in the legs and feet leading to decreased sensation, open leg sores below the knee, peripheral neuropathy.

This product is intended for use by adults.

Please consult a healthcare professional before use if you: Have had blood clots already/have a family history of blood clots, suffer from or have had treatment for cancer, blood diseases, receive treatment for heart failure or circulation problems, have had recent surgery especially on the hips or knees or have an inherited clotting tendency. **Warning:** Do not fold or roll down. If legs develop redness, pain or irritation during use, discontinue use and consult a healthcare professional. Fingernails, oils or greasy ointments may damage the fabric and reduce its life. Do not use if fabric shows signs of degradation. Any serious incident that has occurred in relation to this device should be reported to G&N and the Competent Authority in your country.

Washing Instructions

- 90% Nylon, 10% Elastane
- Machine wash do not exceed 40°C/104°F
 - No bleach products should be used
 - Do not iron
 - Do not dry clean
 - Do not tumble dry

EC REP CS Lifesciences Europe Limited,
The Black Church, St Mary's Place, Dublin 7,
D07 P4AX Ireland

G&N Medical, Stane Street, Slinfold, Horsham,
West Sussex, RH13 0GN
Office: +44 (0) 1403 799190
www.gandn.com | www.fitlegs.com

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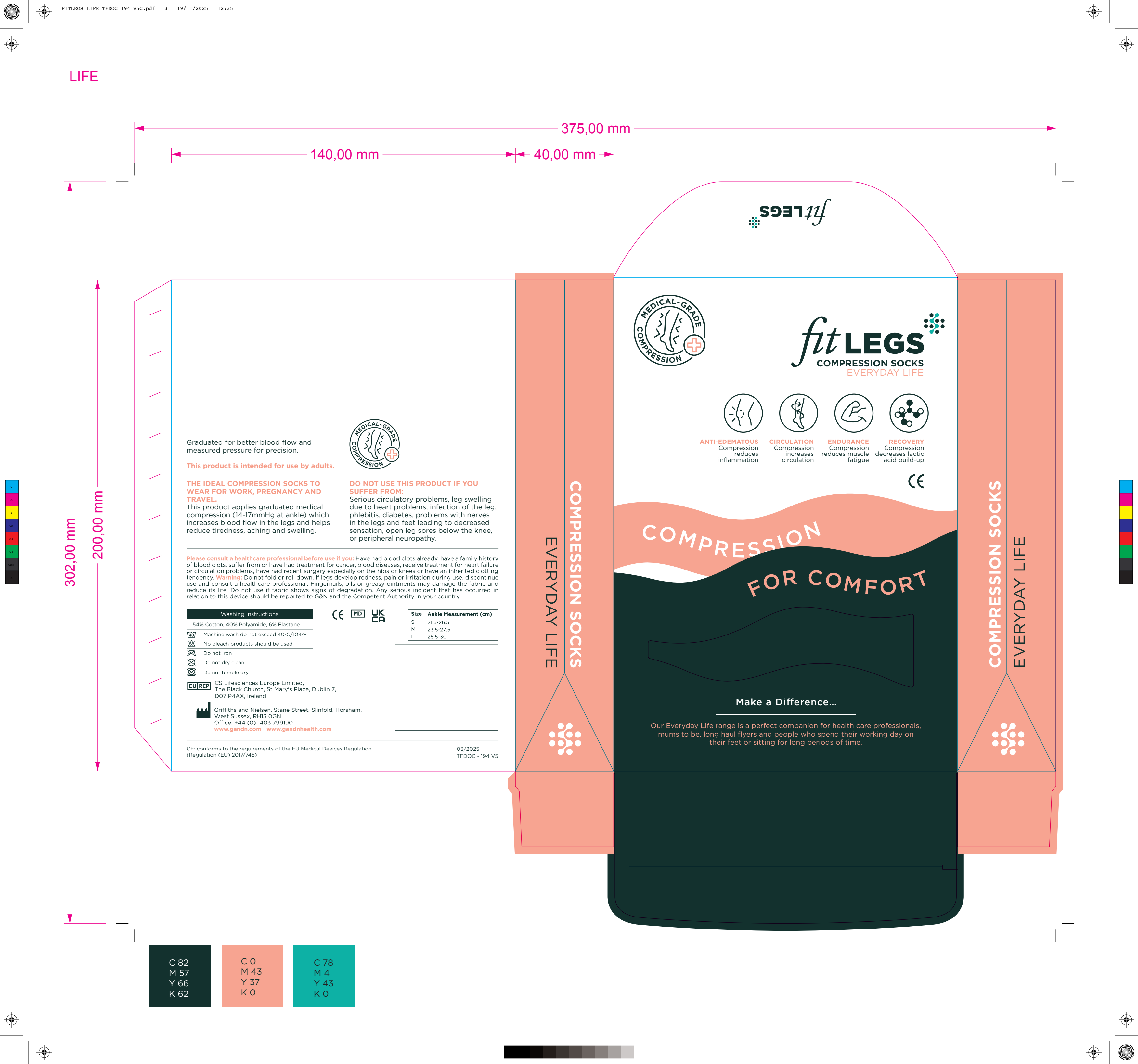
302,00 mm

200,00 mm

Graduated for better blood flow and measured pressure for precision.

This product is intended for use by adults.

THE IDEAL COMPRESSION SOCKS TO WEAR FOR WORK, PREGNANCY AND TRAVEL.

This product applies graduated medical compression (14-17mmHg at ankle) which increases blood flow in the legs and helps reduce tiredness, aching and swelling.

DO NOT USE THIS PRODUCT IF YOU SUFFER FROM:
Serious circulatory problems, leg swelling due to heart problems, infection of the leg, phlebitis, diabetes, problems with nerves in the legs and feet leading to decreased sensation, open leg sores below the knee, or peripheral neuropathy.

Please consult a healthcare professional before use if you: Have had blood clots already, have a family history of blood clots, suffer from or have had treatment for cancer, blood diseases, receive treatment for heart failure or circulation problems, have had recent surgery especially on the hips or knees or have an inherited clotting tendency. **Warning:** Do not fold or roll down. If legs develop redness, pain or irritation during use, discontinue use and consult a healthcare professional. Fingernails, oils or greasy ointments may damage the fabric and reduce its life. Do not use if fabric shows signs of degradation. Any serious incident that has occurred in relation to this device should be reported to G&N and the Competent Authority in your country.

Washing Instructions

54% Cotton, 40% Polyamide, 6% Elastane

- Machine wash do not exceed 40°C/104°F
- No bleach products should be used
- Do not iron
- Do not dry clean
- Do not tumble dry

CS Lifesciences Europe Limited,
The Black Church, St Mary's Place, Dublin 7,
D07 P4AX, Ireland

Griffiths and Nielsen, Stane Street, Slinfold, Horsham,
West Sussex, RH13 0GN
Office: +44 (0) 1403 799190
www.gandn.com | www.gandnhealth.com

CE: conforms to the requirements of the EU Medical Devices Regulation
(Regulation (EU) 2017/745)

03/2025
TFDOC - 194 V5

C 82
M 57
Y 66
K 62

C 0
M 43
Y 37
K 0

C 78
M 4
Y 43
K 0

RECOVERY

Graduated for better blood flow and measured pressure for precision

This product is intended for use by adults.

THE IDEAL COMPRESSION SOCKS TO WEAR AFTER EXERCISE.

This product applies graduated medical compression (14-17mmHg at ankle) which improves blood flow in the legs and helps reduce tiredness, aching and swelling which enables you to exercise for longer and recover faster.

DO NOT USE THIS PRODUCT IF YOU SUFFER FROM:

Serious circulatory problems, leg swelling due to heart problems, infection of the leg, phlebitis, diabetes, problems with nerves in the legs and feet leading to decreased sensation, open leg sores below the knee, or peripheral neuropathy.

Please consult a healthcare professional before use if you: Have had blood clots already, have a family history of blood clots, suffer from or have had treatment for cancer, blood diseases, receive treatment for heart failure or circulation problems, have had recent surgery especially on the hips or knees or have an inherited clotting tendency. **Warning:** Do not fold or roll down. If legs develop redness, pain or irritation during use, discontinue use and consult a healthcare professional. Fingernails, oils or greasy ointments may damage the fabric and reduce its life. Do not use if fabric shows signs of degradation. Any serious incident that has occurred in relation to this device should be reported to G&N and the Competent Authority in your country.

Washing Instructions	
50% Bamboo, 44% Polyamide, 6% Elastane	
	Machine wash do not exceed 40°C/104°F
	No bleach products should be used
	Do not iron
	Do not dry clean
	Do not tumble dry

EU REP CS Lifesciences Europe Limited,
The Black Church, St Mary's Place, Dublin 7,
D07 P4AX, Ireland


Griffiths and Nielsen, Stane Street, Slinfold, Horsham,
West Sussex, RH13 0GN
Office: +44 (0) 1403 799190
www.gandn.com | www.gandnhealth.com

CE: conforms to the requirements of the EU Medical Devices Regulation
(Regulation (EU) 2017/745)

03/2025
TFDOC - 278 V3

Size	Ankle Measurement (cm)
S	21.5-26.5
M	23.5-27.5
L	25.5-30

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fit LEGS
COMPRESSION SOCKS
RECOVERY

ANTI-EDEMATOUS
Compression
reduces
inflammation

CIRCULATION
Compression
increases
circulation

ENDURANCE
Compression
reduces muscle
fatigue

RECOVERY
Compression
decreases lactic
acid build-up

COMPRESSION
FOR RECOVERY

Make a Difference...

Our Recovery range is a perfect companion for people who want
to improve recovery after strenuous exercise.

COMPRESSION SOCKS

COMPRESSION SOCKS

RECOVERY

SPORT

Graduated for better blood flow and measured pressure for precision.

This product is intended for use by adults.

THE IDEAL COMPRESSION SOCKS TO WEAR FOR EXERCISE.

This product applies graduated compression (18-24mmHg at ankle) which improves blood flow in the legs, enabling you to exercise for longer and improve performance.

DO NOT USE THIS PRODUCT IF YOU SUFFER FROM:

Serious circulatory problems, leg swelling due to heart problems, infection of the leg, phlebitis, diabetes, problems with nerves in the legs and feet leading to decreased sensation, open leg sores below the knee, or peripheral neuropathy.

Please consult a healthcare professional before use if you: Have had blood clots already, have a family history of blood clots, suffer from or have had treatment for cancer, blood diseases, receive treatment for heart failure or circulation problems, have had recent surgery especially on the hips or knees or have an inherited clotting tendency. **Warning:** Do not fold or roll down. If legs develop redness, pain or irritation during use, discontinue use and consult a healthcare professional. Fingernails, oils or greasy ointments may damage the fabric and reduce its life. Do not use if fabric shows signs of degradation. Any serious incident that has occurred in relation to this device should be reported to G&N and the Competent Authority in your country.

Washing Instructions	
95% Polyamide, 5% Elastane	
	Machine wash do not exceed 40°C/104°F
	No bleach products should be used
	Do not iron
	Do not dry clean
	Do not tumble dry

EU REP CS Lifesciences Europe Limited,
The Black Church, St Mary's Place, Dublin 7,
D07 P4AX, Ireland

Griffiths and Nielsen, Stane Street, Slinfold, Horsham,
West Sussex, RH13 0GN
Office: +44 (0) 1403 799190
www.gandn.com | www.gandnhealth.com

CE: conforms to the requirements of the EU Medical Devices Regulation
(Regulation (EU) 2017/745)

03/2025
TFDOC - 195 V5

Size	Ankle Measurement (cm)
S	21.5-26.5
M	23.5-27.5
L	25.5-30

fitLEGS
COMPRESSION SOCKS
SPORT

ANTI-EDEMATOUS
Compression
reduces
inflammation

CIRCULATION
Compression
increases
circulation

ENDURANCE
Compression
reduces muscle
fatigue

RECOVERY
Compression
decreases lactic
acid build-up

COMPRESSION
FOR PERFORMANCE

Make a Difference...

Our Sport range is a perfect companion for people who want to improve performance, increase endurance and reduce the risk of injury during exercise.

SPORT

COMPRESSION SOCKS

SPORT

GRADUATED FOR BETTER BLOOD FLOW AND MEASURED PRESSURE FOR PRECISION.

This product is intended for use by adults.

THE IDEAL COMPRESSION SOCKS TO WEAR FOR WORK, PREGNANCY AND TRAVEL.

This product applies graduated medical compression (14-17mmHg at the ankle) which increases blood flow in the legs and helps reduce tiredness, aching and swelling. Made with a soft bamboo blend for comfortable everyday wear.

DO NOT USE THIS PRODUCT IF YOU SUFFER FROM:

Serious circulatory problems, leg swelling due to heart problems, infection of the leg, phlebitis, diabetes, problems with nerves in the legs and feet leading to decreased sensation, open leg sores below the knee, or peripheral neuropathy.

Please consult a healthcare professional before use if you: Have had blood clots already, have a family history of blood clots, suffer from or have had treatment for cancer, blood diseases, receive treatment for heart failure or circulation problems, have had recent surgery especially on the hips or knees or have an inherited clotting tendency. **Warning:** Do not fold or roll down. If legs develop redness, pain or irritation during use, discontinue use and consult a healthcare professional. Fingernails, oils or greasy ointments may damage the fabric and reduce its life. Do not use if fabric shows signs of degradation. Any serious incident that has occurred in relation to this device should be reported to G&N and the Competent Authority in your country.

Washing Instructions

54% Viscose Bamboo 40% Polyamide 6% Elastane

- Machine wash do not exceed 40°C/104°F
- No bleach products should be used
- Do not iron
- Do not dry clean
- Do not tumble dry

EU REP CS Lifesciences Europe Limited, The Black Church, St Mary's Place, Dublin 7, D07 P4AX, Ireland

Griffiths and Nielsen, Stane Street, Slinfold, Horsham, West Sussex, RH13 0GN
Office: +44 (0) 1403 799190
www.gandn.com | www.gandnhealth.com

CE: conforms to the requirements of the EU Medical Devices Regulation (Regulation (EU) 2017/745)

TFDOC-346

Size	Ankle Measurement (cm)
S	21.5-26.5
M	23.5-27.5
L	25.5-30

COMPRESSION SOCKS
SUSTAINABLE BAMBOO

COMPRESSION SOCKS
SUSTAINABLE BAMBOO

COMPRESSION SOCKS
SUSTAINABLE BAMBOO

- ANTI-INFLAMMATION**
Compression reduces swelling
- CIRCULATION**
Compression increases circulation
- ENDURANCE**
Compression reduces muscle fatigue
- RECOVERY**
Compression decreases lactic acid build-up

Make a Difference...

Everyday Life Naturals is our modern, premium take on everyday compression — crafted with a soft bamboo blend for a more sustainable yarn. Ideal for health care professionals, mums-to-be, long-haul flyers and anyone on their feet or sitting for long periods of time...